




# QUICK START GUIDE



## 6.3 ColostrFIT Checklist Colostrum bank available


**1** Start of calving

**2** ColostrMAT is ready on Standby

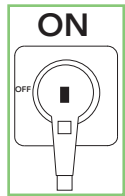


**3** Calf is born

**4** Put ColostrBAG in the ColostrMAT and use Profile 3 to thaw colostrum

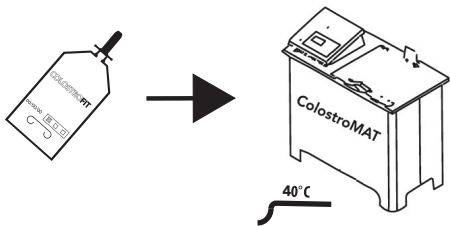

**Calf**

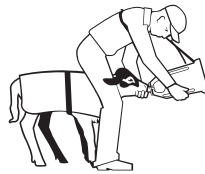


Feed calf as soon as possible (within 4 hours) with ColostrBAG



Dispose ColostrBAG




**Cow**

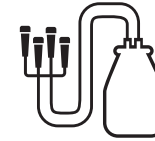


Milk cow as soon as possible



Check the colostrum quality (Colostrometer in the green range; Refractometer >22 Brix%)

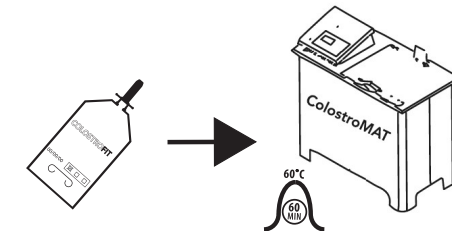

Fill colostrum in ColostrBAG and pasteurize with Profile 2

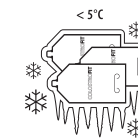

Freeze and store pasteurized colostrum